



Application Guide *LiquiBand® Exceed™*

LiquiBand® Exceed™ Topical Skin Adhesive has a unique, felt applicator tip to facilitate even, consistent application in three simple steps.


✓ STEP ONE

- ▶ While holding upright, squeeze the applicator wings until a cracking sound is heard.



✓ STEP TWO

- ▶ Invert the tip and gently squeeze wings to prime the device. When the tip turns violet, LiquiBand® Exceed™ is ready to use.
- ▶ Ensure that wound is thoroughly clean and dry prior to closure.

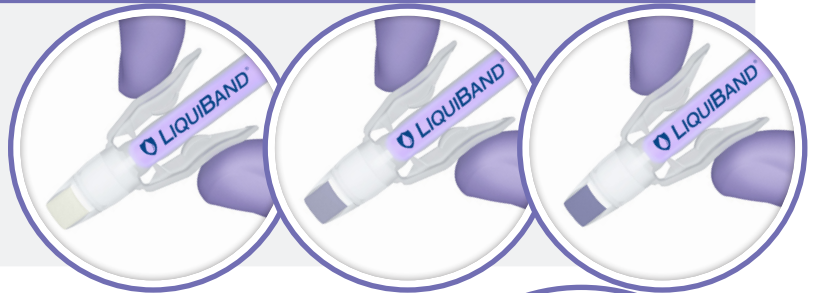

✓ STEP THREE

- ▶ While holding the wound edges together, apply an even coat of LiquiBand® Exceed™ over the length of the wound.
- ▶ Continue to hold the wound edges together for 60 seconds.



LiquiBand® Exceed™

Size	Qty Per Box	Product Code
0.8g	6	72014002

1 If using on the eye area, adjust the patient's position. Use petroleum jelly to create a barrier close to the wound, and use surgical gauze to protect the patient's eyes.



T 1-855-GLUES-77
E info@liquiband.com
W liquiband.com