


## LiquiBand® Surgical S

The high strength Butyl-based cyanoacrylate offering the versatility of two application tips.



### ✔ STRONG AND SECURE

- ▶ Butyl / Octyl blend, providing a strong primary closure and a flexible dressing
- ▶ Effective microbial barrier against gram positive, gram negative and fungal microbes

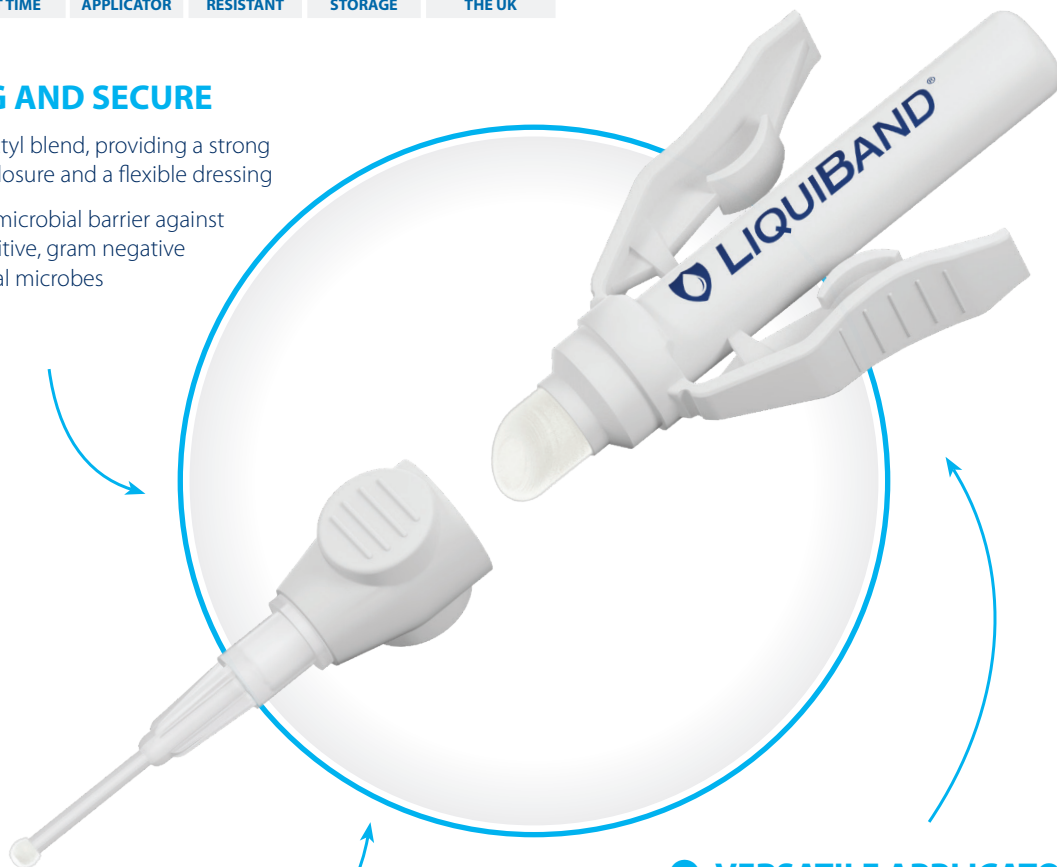

### ✔ FAST AND PRECISE

- ▶ Quick 10-second dry time
- ▶ Controlled placement of adhesive


### ✔ VERSATILE APPLICATOR

- ▶ Winged applicator allows easy and safe priming and application
- ▶ Flow Control tip for maximum control
- ▶ Deer-foot tip for broad application

# Application Guide *LiquiBand® Surgical S*


## ✓ STEP ONE

- ▶ Ensure that wound is thoroughly clean and dry prior to closure.
- ▶ While holding upright, squeeze the applicator wings until a cracking sound is heard.

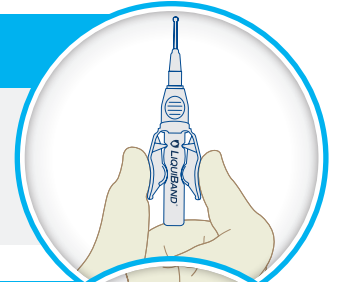


## ✓ STEP TWO

- ▶ Invert the tip and gently squeeze wings to prime the device.
- ▶ When the tip fills with adhesive, LiquiBand® Surgical S is ready to use.

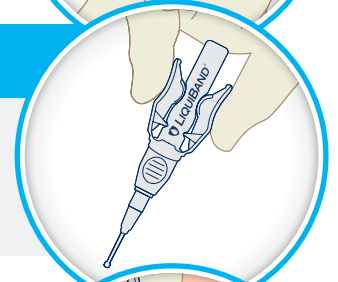


## ✓ STEP THREE

- ▶ While holding the wound edges together, apply an even coat of LiquiBand® Surgical S over the length of the wound.
- ▶ Continue to hold the wound edges together for 10 seconds.

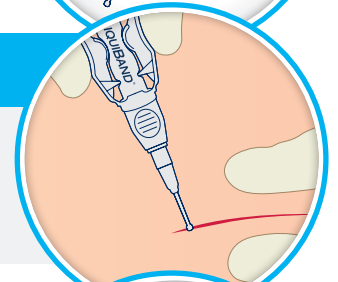


### Relative strengths of topical cyanoacrylates and sutures<sup>1</sup>

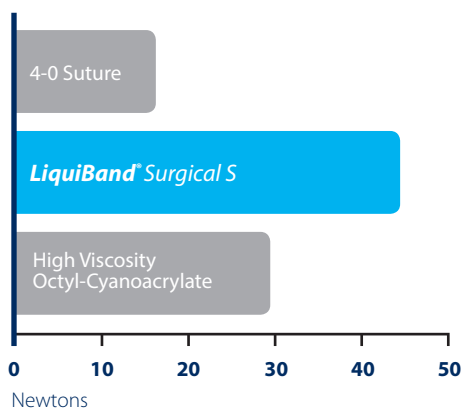

References 1. Data on file at Advanced Medical Solutions (Plymouth) Ltd.

## ✓ STEP FOUR (Optional)

- ▶ Twist the precision tip anti-clockwise and pull off the precision tip
- ▶ The deer-foot tip allows for a broad application

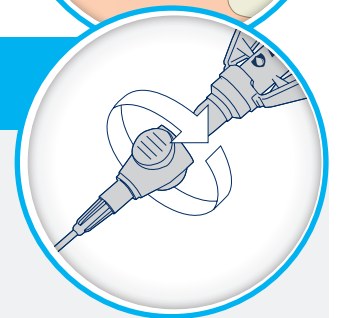

### *LiquiBand® Surgical S*

Size	Qty Per Box	Product Code	NHS SC Code
0.8g	6	LBS-S0001	FVF 014